Acqua Ossigenata 12V

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Acqua Ossigenata 12V

Ready-to-use hydrogen stabilized peroxide solution for root canal irrigation

Composition

3.6% w/w stabilized aqueous solution of hydrogen peroxide.

Properties

12 V hydrogen peroxide is characterized by its high oxidizing power. When used alternately with sodium hypochlorite, it actively releases oxygen which flushes organic and mineral residues out of the root canals.

Indications

Ready-to-use solution for root canal irrigation during shaping.

Instructions for use

Pull out the tamper-evident cap. Attach the enclosed perforator to a syringe. Draw up a sufficient volume of solution. Remove the perforator from the syringe and replace it with a blunt irrigation needle. Flush the canal alternately with sodium hypochlorite and rinse with physiological saline or distilled water to complete treatment.

Caution: do not use the perforator for irrigation.

Precautions for use

To inject irrigating solutions, it is recommended to use specifically designed blunt endodontic irrigation needles, to prevent accidental extrusion of irrigating solution beyond the apex.

Use a rubber dam.

N.B.: a slight deformation of the bottle seal does not compromise the use of the product.

To ensure the correct storage of the product, it is recommended not to remove the metal ring completely but only the tamper-evident cap.

Warnings

FOR EXTERNAL USE ONLY – DO NOT INJECT – DO NOT SWALLOW.

Slightly irritating to eyes and skin.

Keep out of the reach of children.

In case of contact with skin or eyes, rinse immediately with plenty of water.

Wear suitable protective clothing and eye/face protection.

Avoid release to the environment.

Storage

Do not store above 30°C.

Presentation

5 x 50 g plasticized glass bottles + 1 perforator.

For professional use only in the practice of dentistry.

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Giovanni Ognà & figli S.p.A.