FR

## COMPOSITION

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UK

## COMPOSITION

Polyester/cotton cords containing aluminium chloride hexahydrate and lidocaine base.

## INDICATION

Retraction of free gingiva before impression taking.

## CONTRAINDICATIONS

Do not use in children under 4 years of age.  
Do not use in patients with a history of allergy to lidocaine (amide local anaesthetics) or to aluminium and aluminium salts.

## INSTRUCTIONS FOR USE

If temporary gingival retraction is needed for several teeth, treat one tooth at a time.

Pull the cord out of the bottle and cut the required length: the tooth crown size may vary from 2 to 6 cm or even 7 cm.  
Cut longer pieces of cords than required so that when the cord is wound up

around the tooth, the ends overlap. This makes the removal of the cord easier.

Place the cord in the sulcus (gingival crevice) using a metallic instrument with a blunt end so as not to injure the epithelial attachment. Start with the inter-proximal space because it is where the sulcus is usually the deepest. Then proceed clockwise until the cord has been placed all around the tooth.

Once inserted, keep the retraction cord in place for about 6-7 minutes, up to 10 minutes. Remove it just before impression-taking except if you are using the double impression technique. Slightly moisten the cord to make its removal easier and less painful.

## SPECIAL WARNINGS AND PRECAUTIONS FOR USE

Flammable.

- After taking the impression, examine the sulcus thoroughly to make sure that the epithelial attachment has not been torn and that no fragments of the retraction cord have been left in the gingival crevice, which as any foreign body could cause acute local inflammation.
- The cord should only be used on healthy tissue, free from lesions, bleeding and infection.

## STORAGE

Keep the product dry and away from light.

## PRESENTATIONS

- 15 ml bottle with grommet cap containing 4 metres of thin cord (0.6 mm diameter).
- 15 ml bottle with grommet cap containing 4 metres of medium cord (0.8 mm diameter).
- 15 ml bottle with grommet cap containing 3 metres of thick cord (1.7 mm diameter).

*For professional dental use only.*

**Distributor:** SEPTODONT Ltd.  
Units R & S – Orchard Business Center  
St Barnabas Close Allington, Maidstone  
Kent ME16 0JZ - UK

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