# Vaposept

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# Vaposept

Disinfection by spraying in dental surgery

## Microbiological properties

Bactericidal : standards NF T 72-152/EN 1040, NF T 72-171 and NF T 72-190.  
Fungicidal: active on Candida albicans (standard NF T 72-202/EN 1275), Aspergillus niger (standard T 72-300) and Aspergillus fumigatus (standard T 72-301).  
Active on HIV-1 virus (AIDS) -1987 and HBV virus (Hepatitis B).  
Tuberculicidal : active on Koch's Bacillus (Mycobacterium tuberculosis)

## Composition

Aldehydes, surfactant, perfumed hydro-alcoholic excipient.

## Directions for use

Solution ready for use. Disinfection is only possible on clean surfaces.  
Spray directly onto the area which must be completely and uniformly wetted. Let dry. Do not wipe or rinse.

## Warnings and precautions for use

For external use only. Do not ingest. To be used undiluted.  
Do not spray onto surfaces sensitive to alcohol.  
Incompatibility with ammoniac and ammoniacal detergents.

**Contains formaldehyde.** May cause allergic reactions.

**Specific risk :** Flammable.

**Safety advice :** Keep out of the reach of children.

Keep container tightly closed. Keep away from sources of ignition - No smoking. Do not breathe spray. Avoid contact with eyes. In case of accident or if you feel unwell, seek medical advice immediately (show the label where possible). Keep only in the original container.

## Presentations

2 plastic bottles containing 500 ml and 1 dispenser.  
5 l. refill pack with tap.

## Instruction for refilling

- 1 - Unscrew the cap of the refill pack.
- 2 - Screw the tap making sure it is perfectly turned off (F).
- 3 - Lay the container down so that the tap might be presented at the bottom and in front.
- 4 - Place the container to be filled under the tap.
- 5 - Turn the tap on by putting the handles on position (O). Regulate the flow of air by turning the upper handle to the right.

*For professional use only in the practice of dentistry.*

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