


Alveogyl

Dry socket? Ease the pain

Alveogyl is the ideal dry socket dressing, rapidly easing your patient's pain and convenient for you to use. Alveogyl may also be used as a post-extraction dressing.

Alveogyl is a one-step, self-eliminating treatment requiring no suturing and no special attention other than observation of the healing process. It provides a soothing effect on the alveolar tissues thus helping to rapidly alleviate the pain. Its fibrous consistency, due to the Penghawar fibers, allows an easy filling of the socket and a good adherence to the alveolus.

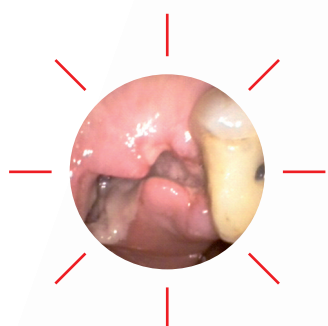

To order Alveogyl,
contact your local dental dealer

Alveogyl characteristics

Features	Advantages	Benefits
Contains Penghawar fibers	Fibrous consistency	Ensures easy filling of the socket and a good adherence to the alveolus during the entire healing process
Dry socket dressing	Provides a soothing effect on the alveolar tissues throughout the healing process	Helps to rapidly alleviate the pain
No suturing required	No special attention other than observation of the healing process	Convenient and easy to use
Self eliminating treatment	One-step procedure	No need for recall appointments

Indications

Alveogyl may be used as a post-extraction dressing following a difficult or traumatic extraction in patients having a history of dry sockets.

Instructions for use

- Take a small pellet (about 0.20 g) of Aveogyl paste
- Place it gently into the prepared dental socket
- Do not suture

Product Information



Alveogyl

10 g Jar

