Endomet Plain

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Composition

Powder :

Thymol iodide, zinc oxide, barium sulfate, excipient.

Liquid :

Eugenol, excipient.

Indications

Root canal filling and sealing.

The paste may be used alone or combined with a point.

Properties

ENDOMET PLAIN is a zinc oxide-eugenol based root canal sealer to which various ingredients have been added viz. :

- thymol iodide and barium sulfate to increase its radiopacity and to reduce its retraction tendency.

Instructions

Mix carefully for approximately 30 seconds 2 level measuring spoonfulls of powder with 4/5 drops of liquid so as to obtain a thick mix which can easily be introduced into the canal with a suitable root canal filler, or in conjunction with a silver, acrylic or gutta percha point.

Warnings and precautions for use

ENDOMET liquid is irritating to skin.

The use of a nonputrescible paste does not, however, allow for improper reaming and preparation of the canals. Should a small amount of paste be accidentally extruded beyond the apex, the product will be slowly absorbed in these well irrigated tissues before hardening. On the other hand, the product will not be notably resorbed within the canal where the irrigation is very poor.

Presentation

Supplied as a kit containing :

- One 14 g of powder,
- One 10 ml bottle of liquid,
- A measuring spoon,
- A mixing block.

For professional use only in the practice of dentistry.

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