F

COMPOSITION

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INDICATIONS

Periodontology : protective gingival dressing for localized periodontitis.
Operative dentistry : temporary dressing. In the case of cervical caries, gingival retraction before insertion of filling.

PROPERTIES

Self-hardening plastic paste containing fibres in its mass. This paste can also be combined, as a neutral medium, to some medicines so that they might be kept in place easily at the gingiva, the tooth or the dental alveolus level.

INSTRUCTIONS FOR USE

General information : to prevent contamination of the contents, always use a dry sterile spatula. The mass should not be wetted throughout manipulation. The jar should be tightly closed after use. Working time in the mouth is only 2 or 3 minutes following application. Setting time is half an hour.

Periodontology :

It is a protective dressing after gingivectomy : after swabbing the area, a mass of the length and diameter required to cover this area will be prepared, then introduced into the interproximal spaces in order to stabilize the mass. Ensure that the dressing does not extend beyond the immediate area to be protected.

It is recommended for the patient's comfort that the dressing be renewed every 2 to 5 days.

Dental surgery :

Temporary filling : especially cavity preparations for inlays or onlays : dry the cavity with warm air and fill in with SEPTO-PACK. To remove the temporary filling in one piece, use a scaler.

WARNINGS AND PRECAUTIONS FOR USE

It is recommended to coat the walls of the inlay-cavity with a thin layer of vaseline before the application of SEPTO-PACK ; thus, removal of this temporary filling and the subsequent trial of the inlay will be easier.

Thoroughly clean the cavity before permanent sealing of the inlay.

T – Toxic

N – Dangerous for the environment
Contains dibutyl phtalate (DBP)

Very toxic to aquatic organisms, may cause long-term adverse effects in the aquatic environment. Irritating to eyes.

May cause harm to the unborn child.

Possible risk of impaired fertility.

Wear suitable protective clothing and gloves.

In case of insufficient ventilation, wear suitable respiratory equipment.

In case of accident or if you feel unwell, seek medical advice immediately (show the label where possible).

Avoid exposure - obtain special instructions before use.

Avoid release to the environment. Refer to special instructions/safety data sheets.

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