R4+

FR - CHX 2% solution canalaire	2
EN - CHX 2% root-canal solution	3
ES - CHX 2% solución de conducto radicular	4
DE - CHX 2% Wurzelkanallösung	5
IT - CHX soluzione al 2% per canale radicolare	6
PT - Solução CHX 2% de canal radicular	7
NL - CHX 2% wortelkanaaloplossing	8
PL - CHX 2% roztwór kanału korzeniowego	9
EL - CHX 2% διάλυμα ριζικού καναλιού	10
BG - CHX 2% разтвор на коренов канал	11
CS - CHX 2% roztok kořenového kanálu	12
DA - CHX 2% rodkanalopløsning	13
ET - CHX 2% juurekanali lahus	14
HU - CHX 2% gyökércsatorna-oldat	15
LT - CHX 2% sakņu kanāla risinājums	16
LV - CHX 2% šaknies kanalo tirpalas	17
RO - CHX 2% soluție de canal radicular	18
SK - CHX 2% roztok koreňového kanálíka	19
SL - CHX 2% raztopina koreninskega kanala	20
SV - CHX 2% root-canal lösning	21

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**Description:**

R4+ CHX 2% is a chlorhexidine digluconate solution. It is intended for the irrigation and debridement of root canals after instrumentation in endodontic treatment. Remove smear layer with appropriate irrigant such as EDTA solution or Citric Acid 12.5%.

For professional use only.

Instruction for use:

1. Use EDTA 17% as a pre-rinse in the canal to remove smear layer.
2. Rinse with Sodium Hypochlorite (NaOCl).
3. Rinse out the NaOCl with sterile water or anesthetic solution
4. Fill syringe with R4+ CHX 2% (5-10 ml recommended). Attach irrigation tip securely to syringe.
5. Continuously irrigate for a minimum of 2 minutes. (10 – 20 ml per canal recommended).
6. Remove and dispose of syringe and irrigating tip.
7. Dry and obturate the root canal.

Precautions:

1. Read MSDS for material handling and safety information.
2. Do not ingest, do not inject.
3. Not to be used as an oral rinse.

Warnings:

1. Products containing chlorhexidine must not be mixed with products containing sodium hypochlorite to prevent formation of a reddish-brown low solubility precipitate.
2. Keep out of the reach of children.
3. Avoid contact with skin and eyes.

Storage:

Store at room temperature, dry area, away from heat and direct sunlight.

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**Inter-Med, Inc.**